

TEOXANE

TEO



THE
EXPERT
CASE
BOOK #2

DYNAMIC
VOLUMIZING
& LIFTING EFFECT

WITH RHA 4 & ULTRA DEEP

12 CLINICAL CASES

with RHA 4
& ULTRA DEEP

1 - 2
DR. BONG
p. 6 to 9

DR. DJALAEI
p. 10

3

4
DR. INGALLINA
p. 12

DR. MATANIS
p. 14

5

6-7
DR. PALLOVÁ
p. 16 to 19

DR. REDKA-
SWOBODA
p. 20

8

9-10
DR. RICCIUTI
p. 22 to 25

DR. SANTANA
p. 26

11

12
DR. SEBAGH
p. 28

"The options to use ULTRA DEEP (UD) and RHA 4 separately or combined are offering me the ideal freedom of treatment. UD is a highly viscous HA which offers impressive results in creating and restoring volumes. When it comes to sculpting and contouring the face, I tend to inject UD in the deep fat compartments, supra-periosteally. Its high crosslink percentage means only a minimal amount of UD is required for effective volume restoration, which also provides for long lasting results.

RHA's unique molecular structure offers this range of HA superior versatility. Although it can be used in various parts of the face, RHA 4 shines when it is used to re-sculpt facial contours and restore original curves found in the face. It is the ideal HA for the perfect remodeling of the face due to its high volume creating capacity. To get the most out of RHA's unique structure that consists mostly of long chain HA, I am inclined to inject RHA 4 in the subdermal plane.

As with the entire range of TEOSYAL products, the results from the combined use of UD and RHA 4 can be seen instantly. When used in conjunction with the right technique, the transformation to the face can be impressive yet very natural."

Dr. Kieren Bong, UK



PATIENT INFORMATION

Gender: Female
Age: 43 years old
Smoking habits: 10 cigarettes per day
Phototype: III
Photo-ageing: Advanced


Skin quality: Dry, dull and rough skin, diffused dyschromia and wrinkles, teleangiectasias
Previous treatment: Calcium hydroxylapatite, 3 years ago; she was disappointed by the results

PATIENT NEED

Global rejuvenation after weight loss.


Dr. Ricciuti proposed ULTRA DEEP for the volume restoration and RHA 4 for a lifting effect of the zygomatic-malar area.

STRATEGIES

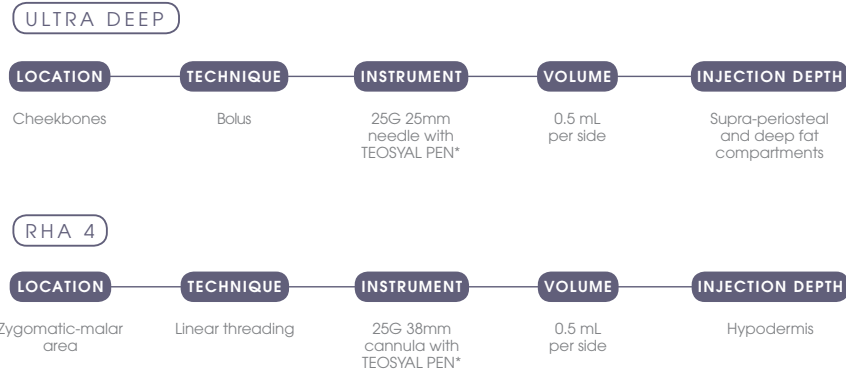


RESULTS

BEFORE

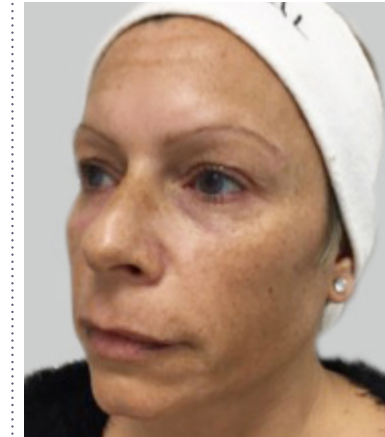

AFTER PROCEDURE

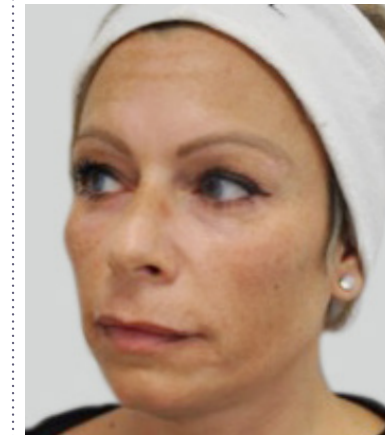

Dr. Francesco Ricciuti, studied and got his degree at the Faculty of Medicine and Surgery in Perugia. Additional Master Degree in Dermatology and Venereology at the University of Pisa. He attended the Master Degree in Dermo-Surgery at the University of Siena and several Master courses in Dermo-Surgery, Dermoscopy, Corrective and Aesthetic Dermatology. He is Dermatologist Medical Director at the Plastic Surgery Unit of Matera Hospital (Basilicate, Italy) where he works in the field of Clinic Dermatology, Dermo-Surgery and psoriasis. He is also an aesthetic medicine freelancer since several years performing dermo-aesthetic treatments using different typology of laser and products like fillers, bio-stimulating cocktails, revitalizing and exfoliating or modulating substances (botulinum toxin).

TEOSYAL® PEN is a non-sterile device class Iia (CE 0434) intended to assist medical practitioners in injection dermal fillers. Manufactured by JuvoPlus in Switzerland. Please refer to your practitioners for more details about your treatment and for further information please read carefully the user guide by asking to your practitioner for the one provided in the box or the recommendations available on www.teosyalpen.com.

PATIENT INFORMATION


Gender: Female
Age: 48 years old
Smoking habits: 10 cigarettes per day
Phototype: III
Photo-ageing: Moderate

Skin quality: Oily and dull skin, mild dyschromia, wrinkles, teleangiectasias
Previous treatment: None


PATIENT NEED

- PRODUCTS: ULTRA DEEP, RHA 4
- PROTOCOL: ULTRA DEEP for the volume restoration; RHA 4 for a lifting effect of the zygomatic-malar area

STRATEGIES

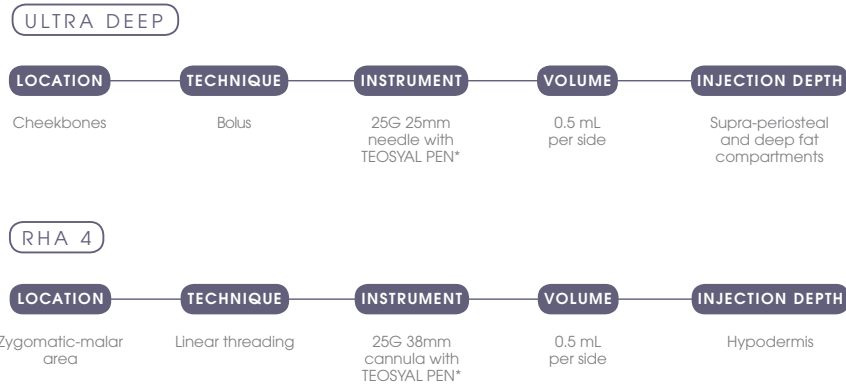

RESULTS

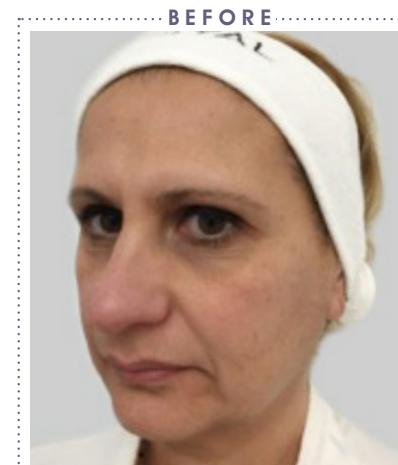

* TEOSYAL® PEN is a motorised and cordless device for hyaluronic acid injections in order to correct wrinkles and depressions. Manufactured by Juvaplus SA. This medical device (class IIa), is a regulated health product which bears, under this regulation, the CE mark (CE0434).



TEOXANE

THE EXCELLENCE OF SWISS SCIENCE

In today's world, beauty is no longer a question of age. Women want their beauty to last into the future.

That's why TEOXANE Laboratories specialise in the complex development of hyaluronic acid injectable gels with researchers driven by the culture of excellence and innovation as well as a total dedication to effectiveness and safety. Thanks to this approach, the hyaluronic acid gels we produce in our Geneva-based laboratories are of the highest quality and purity.

Mrs Valérie Taupin
CEO & Founder, TEOXANE

Teoxane Geneva
Les Charmilles, Rue de Lyon, 105
CH 1203 Genève
teoxane.com